

INSTRUCTIONS FOR USE MONTHLY AND BI-WEEKLY DISPOSABLE CONTACT LENSES

BG	Инструкции за употреба: Ежедневни (за честа смяна) контактни лещи	02
DA	Brugsanvisning : Kontaktlinser til daglig brug (hyppig udskiftning)	03
DE	Gebrauchsanweisung für : Kontaktlinsen zum täglichen Tragen (regelmäßiges Wechseln) ..	04
EL	Οδηγίες χρήσης : Φακοί επαφής ημερήσιας χρήσης (συχνή αντικατάσταση)	05
EN	Instructions For Use : Daily Wear (Frequent Replacement) Contact Lenses	06
ES	Instrucciones de uso : Lentes de contacto de uso diario (sustitución frecuente)	07
ET	Kasutusjuhend : Päevaste (sageli vahetatavate) kontaktläätsed	08
FI	Käyttöohjeet : Daily Wear (Usein vaihdettavat) piilolinssit	09
FR	Mode d'emploi : Lentilles de contact souples journalières (remplacement fréquent)	10
IS	Notkunarleiðbeiningar : Daglinsur (Tíð Skipti) snertilinsur	11
IT	Istruzioni per l'uso: Lenti a contatto ad uso giornaliero (a sostituzione frequente)	12
LI	Naudojimo instrukcija : Kasdien nešiojami (dažnai keičiami) kontaktiniai lęšiai	13
LV	Lietošanas instrukcija : Ikdienas lietojuma (biežas nomaiņas) kontaktlēcas	14
NL	Gebruiksaanwijzing: Contactlenzen voor dagelijks gebruik (frequente vervanging)	15
NO	Bruksanvisning : Daily Wear (hyppig bytte) kontaktlinser	16
PL	Kasutusjuhend : Soczewki kontaktowe do codziennego noszenia (częsta wymiana)	17
PT	Instruções de utilização : Lentes de contacto de utilização diária (substituição frequente)	18
SV	Bruksanvisning : Kontaktlinser för endagsbruk (byts ut ofta)	19

Instructions For Use

Daily Wear (Frequent Replacement) Contact Lenses

This instruction for use is intended for users of the contact lenses. Please read carefully and always follow your eye care professional's directions and the instructions.

EN

When used for frequent replacement, your prescriber should recommend a care system that is appropriate for your lens. Carefully read and follow specific directions for use and important safety information for each lens care product. Discard the lens if it becomes damaged or you reach the prescribed wearing period recommended by your eye care professional.

The information and instructions for safety and performance apply only to Daily Wear Soft Contact Lenses. For your eye health, it is important to wear your lenses as prescribed by your eye care professional. It is also important to keep your eye care professional fully aware of your medical history. Your eye care professional will tailor a total program of care based on your specific needs.

He or she will review with you all instructions for how to use the lens properly. The safe and effectiveness of contact lenses depends on proper use.

The light blue color will make it more visible. The Soft Contact Lens with UV blocking may block UV light slightly, but cannot protect eyes as the UV protection equipment.

WARNING: UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. You should continue to use UV absorbing eyewear as directed.

The Soft Contact Lens is daily wear contact lens prescribed for frequent replacement. They are different from the rigid contact as they are more soft and flexible.

WEARING RESTRICTIONS AND INDICATIONS :

Monthly Contact Lenses Indication for use:

The Spherical Contact Lenses is indicated as daily wear and monthly disposable soft contact lens for the correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes who exhibit refractive astigmatism of 1.00D or less where the astigmatism does not interfere with visual acuity.

The Toric for Astigmatism Contact Lenses is indicated as daily wear and monthly disposable soft contact lens for the correction of ametropia (myopia and hyperopia) with astigmatism in aphakic and non-aphakic persons with non-diseased eyes and whose powers are from -12.00 to +8.00D and astigmatic corrections are from -0.75 to -2.25D.

The Multifocal for Presbyopia Contact Lenses is indicated as daily wear daily wear and monthly disposable soft contact lens for the correction of ametropia (myopia and hyperopia) and emmetropia with presbyopia in aphakic and non-aphakic persons with non-diseased eyes and whose powers are from -12.00 to +8.00D with add powers from +0.25 to +2.75D. The lenses may be worn by persons who exhibit astigmatism of 1.00D or less where the astigmatism does not interfere with visual acuity.

The lens may be disinfected using a chemical disinfection system only .

The **Daily Wear Soft Contact Lenses** described in this booklet should be removed from your eyes for routine cleaning and disinfecting as prescribed by your eye care professional.

Bi-Weekly Contact Lenses Indication for use:

The Spherical Contact Lenses is indicated as Bi-Weekly disposable soft contact lens for the correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes who exhibit refractive astigmatism of 1.00D or less where the astigmatism does not interfere with visual acuity.

The Toric for Astigmatism Contact Lenses is indicated as daily wear and Bi-Weekly disposable soft contact lens for the correction of ametropia (myopia and hyperopia) with astigmatism in aphakic and non-aphakic persons with non-diseased eyes and whose powers are from -12.00 to +8.00D and astigmatic corrections are from -0.75 to -2.25D.

The Multifocal for Presbyopia Contact Lenses is indicated as daily wear daily wear and Bi-Weekly disposable soft contact lens for the correction of ametropia (myopia and hyperopia) and emmetropia with presbyopia in aphakic and non-aphakic persons with non-diseased eyes and whose powers are from -12.00 to +8.00D with add powers from +0.25 to +2.75D. The lenses may be worn by persons who exhibit astigmatism of 1.00D or less where the astigmatism does not interfere with visual acuity.

The lens may be disinfected using a chemical disinfection system only .

The **Daily Wear Soft Contact Lenses** described in this booklet should be removed from your eyes for routine cleaning and disinfecting as prescribed by your eye care professional.

CONTRAINDICATIONS (REASONS NOT TO USE)

DO NOT USE Daily Wear Soft Contact Lenses when any of the following conditions exists:

- Acute or sub-acute inflammation or infection of the anterior chamber of the eye
- Any eye disease, injury or abnormality that affect the cornea, conjunctiva, or eyelids
- Severe insufficiency of lacrimal secretion (dry eye)
- Corneal hypoesthesia (reduced corneal sensitivity), if not-aphakic
- Any systemic disease that may affect the eye or be exacerbated by wearing contact lens
- Allergic reactions of ocular surface or adnexa that may be induced or exaggerated by the wearing of contact lenses or use of contact lens solutions
- Allergy to any ingredient, such as mercury or Thimerosal, in a solution which is used to care for the **Daily Wear Soft Contact Lens**
- Any active corneal infection (bacterial, fungal, or viral)
- If eye become red or irritated

WARNINGS

INFECTION WITH POSSIBLE PERMANENT DAMAGE TO VISION COULD RESULT FROM THE FAILURE TO STRICTLY FOLLOW RECOMMENDED DIRECTIONS FOR USE AND LENS CARE PROCEDURES.

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE.

It is essential that you follow the directions of the eye care professional and all labeling instructions for proper use of contact lenses and lens care products, including the lens case. You should be advised of the following instructions for use and warnings pertaining to contact lens wear:

- Use only fresh multi-purpose (contact lens disinfecting) solution each time you soak (store) your lenses.
- Rub and rinse your lenses according to the recommended lens rubbing and rinsing times in the labeling of your multi-purpose solution to adequately disinfect your lenses.
- Never use water, saline solution, or rewetting drops to disinfect your lenses. These solutions will not disinfect your lenses. Rub and rinse contact lens cases with fresh, sterile disinfecting solutions (Never Use Water). Wipe the lens cases with fresh, clean tissue. Replace your lens case according to the directions given you by your eye care professional or the labeling that came with your case.
- Contact lens cases can be a source of bacterial growth. Use of non-sterile solution can lead to severe infection, vision loss or blindness.
- Do not expose your contact lenses to water while you are wearing them. Ask your eye care professional for recommendations about wearing your lenses during any activity involving water.
- Discard any remaining solution after the recommended time period indicated on the bottle of multipurpose solution used for disinfecting and soaking your contact lenses.

EYE PROBLEMS, INCLUDING CORNEAL ULCERS, CAN DEVELOP RAPIDLY AND LEAD TO LOSS OF VISION; IF YOU EXPERIENCE:

- Eye Discomfort
- Excessive Tearing
- Vision Changes
- Loss of Vision
- Eye Redness
- Or Other Eye Problems

YOU SHOULD BE INSTRUCTED TO IMMEDIATELY REMOVE THE LENSES, AND PROMPTLY CONTACT YOUR EYE CARE PROFESSIONAL.

- Daily wear lenses are not indicated for overnight wear, and you should be instructed not to wear lenses while sleeping.
- Please do not wear the contact lens continuously more than 12 hours .
- Consult the eye care professional regarding the use of contact lens in certain atmospheric or environmental conditions that can cause irritation to the eye.
- Consult the eye care professional or doctor if you are pregnant. According to personal physical condition, the dimension of cornea might have some change during pregnancy.
- The lens is immersed in the packaging solution contents boric acid, to prevent microbiological contamination. If you might be pregnant, please talk to your doctor before wearing the contact lens as it contains boric acid.
- Borates, which include boric acid and sodium borate, are classified as Category 1B CMR substances when their concentration exceeds 0.1% by weight. However, these compounds are deemed safe for application when used in compliance with the provided label directions.
- Please flush it with clean saline solution before wearing the contact lens.

PRECAUTIONS

Eye care professionals should carefully instruct you about the following care regimen and safety precautions:

- Different solutions cannot always be used together, and not all solutions are safe for use with all lenses. Use only recommended solutions and follow the Eye care professionals' instruction.
- It is recommended to use chemical disinfection solutions for **Daily Wear Soft Contact Lens**.
- If the lens sticks (stops moving) on the eye, you should be instructed to **immediately** consult your eye care professional.
- Always wash and rinse hands before handling lenses. Avoid contaminating lenses with make-up, aerosol sprays or wearing them near noxious fumes. It is best to put on lenses before putting on makeup.
- Always wash and rinse hands before handling lenses. Avoid to scratches the lenses, causing distorted vision and/or injury of the eye, especially with fingernails.
- Carefully follow the handling, insertion, removal, cleaning disinfecting, storing and wearing instructions in you Instructions for the Daily Wear Soft Contact Lens and those prescribed by the eye care professional.
- Never wear lenses beyond the period recommended by the eye care professional.
- The lens has been sterilized and sealed in packaging until open for use. Do not attempt to re-sterilize the lens in any condition, even if it is unopened or unused. Inappropriate sterilization or re-sterilization may cause package to leak or damage the lens, which may lead to cornea scratches, irritation, sense of foreign object, infection or other possible abnormal conditions when wearing the damaged lens.
- Avoid all harmful or irritating vapors and fumes while wearing lenses.
- Ask the eye care professional about wearing lenses during sporting activities.
- Never use tweezers or other tools to remove lenses from the lens container unless specifically indicated for that use. Pour the lens into the hand.
- Always contact the eye care professional before using any medicine in the eyes.
- Always inform the employer of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that you not wear contact lenses.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of your eyes. You should be instructed as to a recommend follow-up schedule.
- Never to share the lens with other users.
- Wearing Schedule for Monthly contact lenses is 30 days within a month.
- Wearing Schedule for Bi-Weekly contact lenses is 14 days within two weeks.

ADVERSE REACTIONS (PROBLEM AND WHAT TO DO)

You should be informed that the following problems may occur:

- Eyes stinging, burning, itching (irritation), or other eye pain
- Comfort is less than when lens was first placed on eye
- Feeling that something is in the eye such as a foreign body or scratched area
- Excessive watering (tearing) of the eyes
- Unusual eye secretions
- Redness of the eyes
- Reduced sharpness of vision (poor visual acuity)
- Blurred vision, rainbows, halos around objects
- Sensitivity to light (photophobia)
- Dry eyes

If you notice any of above, you should be instructed to:


- **Immediately remove lenses.**
- If the discomfort or problem stops, then look closely at the lens. If the lens is in any way damaged, Discard damaged lens. If your lens has dirt or foreign body on it, or the problem stops and the lens appears undamaged, you should thoroughly clean, rinse, and disinfect the lenses; then reinsert them. After reinsertion, if the problem continues, you should immediately remove the lenses and consult the eye care professional.

When any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization, or iritis may be present. You should be instructed to **keep lens off the eye and seek immediate** professional identification of the problem and prompt treatment to avoid serious eye damage.

WEARING INSTRUCTION

The lens packages are individual and maintained sterilely (Sterilized using steam). DO NOT use if the sterile blister package is opened or damaged.

If the lens appears damaged or deformed, DO NOT use it and use the next lens in the multi-pack.

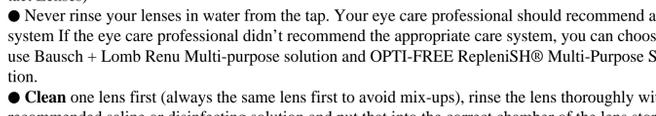

- 1. Placing the Lens on the Eye**
 - Wash and rinse your hands thoroughly and dry them thoroughly with a fiber-free towel, clean and dry before touching the lenses.
 - Verify that the lens is not turned inside out by placing it on your forefinger and checking its profile. The lens should appear a natural, curved, bowl-like shape.
 - Place the lens on the tip of the natural and dry right or left index finger. Place the middle finger of the same hand close to the lashes and pull the lower eyelid down.
 - Use the fingers of the other hand to lift the upper eyelid.
 - Place the lens directly on the eye (cornea) and gently spread your finger away from the lens.
 - Look down and slowly release your lower eyelid.
 - Look straight ahead and slowly release your upper eyelid.
 - Blink gently. If any problems discover, remove your lenses immediately and contact your eye care professional.

- 2. Centering the Lens**

If the lens is not centered on the eye, to center a lens as follow one of the procedures below.

 - a. Close your eyelids and gently massage the lens into place through the closed lids.
 - b. Use finger pressure on the edge of the upper lid or lower lid and gently manipulate the off-centered lens onto the cornea while the eye is open.

- 3. Removing the Lens**

Wash, rinse, and dry your hands thoroughly.

Caution: Always be sure the lens is on the cornea before attempting to remove it. Simply follow the procedures recommended by the eye care professional.

 - a. Pinch Method
 1. Look up; slide the lens to the lower part of the eye using the forefinger.
 2. Pinch the lens between the thumb and forefinger gently.
 - b. Remove the lens.
 - Remove the other lens by following the same procedure.
 - The proper method for disposing of used contact lenses is to throw the lenses in the trash bin.

CARING FOR YOUR LENS (CLEANING, RINSING, STORAGE AND REWETTING / LUBRICATING)

- 1. Basic Instructions:**

It is important to clean, rinse, and disinfect your lenses after each removal, using the care regimen recommended by your eye care professional. You should adhere to a recommended care regimen, avoiding the development of serious ocular complications. For safe contact lens wear, you should know and always practice your lens care routine:

 - Use the recommended system of lens care and carefully follow instructions on solutions labeling.
 - Always follow the schedule prescribed by the eye care professional to remove, clean, rinse, enzyme and disinfect lenses. The use of an enzyme or any cleaning solution does not substitute for disinfection.
 - Disposable and lenses prescribed in a frequent replacement program should be thrown away after the recommended wearing period prescribed by the eye care professional.
 - Once the product is opened, it is allowed to be cleaned 30 times within a month. (Monthly Contact Lenses)
 - Once the product is opened, it is allowed to be cleaned 14 times within 2 weeks. (Bi-Weekly Contact Lenses)
 - Never rinse your lenses in water from the tap. Your eye care professional should recommend a care system If the eye care professional didn't recommend the appropriate care system, you can choose to use Bausch + Lomb Renu Multi-purpose solution and OPTI-FREE RepleniSH® Multi-Purpose Solution.
 - Clean one lens first (always the same lens first to avoid mix-ups), rinse the lens thoroughly with recommended saline or disinfecting solution and put that into the correct chamber of the lens storage case. Then repeat the procedure for the second lens.
 - After cleaning, disinfect lenses using the system recommended by your eye care professional. Follow the instructions provided in the disinfection solution labeling.
 - To store lenses, disinfect and leave them in the closed/unopened case until ready to wear.
 - Prolonged period of drying may damage the lenses. Follow the lens care directions for care for a dried out (dehydrated) lens if the lens surface does become dried out.
 - Your eye care professionals may recommend a lubricating/rewetting solution for your use.

- 2. Chemical disinfection:**
 - Clean the contact lenses with a recommended cleaning solution and thoroughly rinse them with a multi-purpose solution or sterile saline solution.
 - When using hydrogen peroxide lens care systems, lenses **must be neutralized** before wearing. Follow the recommendations on the hydrogen peroxide solution labeling.
 - Lenses that are chemically disinfected may absorb ingredients from the disinfecting solution which may be irritating to your eyes. A thorough rinse in fresh sterile saline solution prior to placement on your eye should reduce the potential for irritation.
 - **Do not heat the disinfection solution and lenses.**

- 3. Lens case cleaning and maintenance**

Lens cases should be emptied, cleaned, rinsed with solutions recommended by the lens case manufacturer, and allowed to air dry. Lens cases should be replaced at regular intervals as recommended by the lens case manufacturer or your eye care professional.

- 4. Care of a dried out (dehydrated) lens**

It is any **Daily Wear Soft Contact Lens** is exposed to air for 15 minutes while off your eye, it may become dry and brittle. Please discard the dehydrated lens and use a fresh new one.

5. Emergencies

If chemicals of any kind (household products, gardening equipment, laboratory chemicals, etc.) are splashed into your eyes, you should: **Flush eyes with tap water and immediately contact your eye care professional or visit a hospital emergency room without delay.**

REPORTING OF ADVERSE REACTIONS (Side Effects)

Any incidents should be reported to the manufacturer and/or its authorized representative and/or to your national authority.

It is essential that you follow your eye care professional's directions regarding this important step of building up your wear time.

SYMBOLS KEY

The following symbols may appear on the label or packaging.

Symbol	Definition	Symbol	Definition
	Consult Instructions for Use		Temperature limit
	Caution		Do not use if package is damaged
	Manufacturer		UV blocking
	Use-By date (expiration date)		CE-mark and Identification Number of Notified Body
	Batch code		Authorized representative in the European Community
	Sterilized using steam		Medical device
	Diameter (the unit is millimeter)		Unique device identifier
	Base curve (the unit is millimeter)		Contains hazardous substances
	Diopter (lens power)		Do not sterilize
	Cylinder		Date of manufacture
	Axis		Single sterile barrier system
	Highest near addition that can be corrected		Importer
	"low" near ADD lens (up to +1.25D)		"medium" near ADD lens (up to +2.00D)
	"high" near ADD lens (above +2.75D)		

EC REP

Emergo Europe B.V.
 Westervoortsedijk 60, 6827 AT Arnhem, The Netherlands.

Visco Technology Sdn. Bhd.
 2686, Jalan Todak, Seberang Jaya, 13700 Prai, Penang, Malaysia
 www.viscovision.com.tw/en/

Version: 1.0 Date: May 2023

